


First steps with your Vivatmo pro



- ① Handheld
- ② Charging cradle
- ③ On/Off button
- ④ Touchscreen
- ⑤ Power adapter port

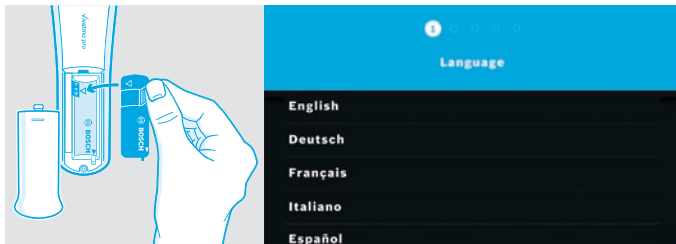

1. Insert the rechargeable battery in the battery compartment on the back of the handheld.
2. Connect the basestation with the power supply.
3. Remove the protective cap and place the handheld on the charging cradle for a few hours.
4. Switch on the basestation by pressing the ON/OFF button
5. Follow the instructions on the Vivatmo pro screen.



Please consider the following before using Vivatmo pro:

Take time to carefully read the instructions for use before using the measurement system. This quick guide does not replace the instructions for use.

Inspect the basestation, the handheld, the disposable mouthpiece and its pouch before usage. If you notice any damage, do not use the device or accessories.



Bosch Healthcare Solutions GmbH
Stuttgarter Str. 130
D-71332 Waiblingen
Germany



F09G100424_01
Latest change: 2019-06-10



www.vivatmo.com

Not available for commercial sale in the United States.

Quick guide

vivatmo
pro


Vivatmo pro - Screen



- ① Measurement orders from electronic patient record
- ② Manage patient measurements and orders
- ③ Charging status handheld battery
- ④ Status of handheld connection
- ⑤ Start measurement
- ⑥ Settings menu/logout


Using Vivatmo pro mobile

Stand-alone measurements can be used when the handheld has no connection to the basestation, when the basestation is out of range or switched off.

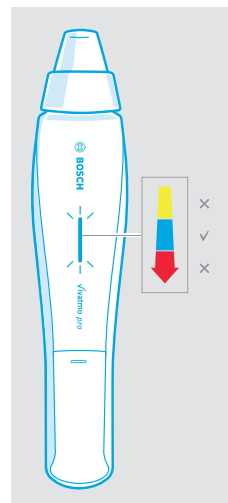

1. Check whether the Vivatmo pro handheld is switched on and attach a new disposable mouthpiece.
2. Press the **Enter** button and wait for the handheld to finish regeneration.
3. When the display shows the **Breathing** icon, start the measurement.
4. The color of the LED provides feedback on the correct breathing strength:
Yellow: too weak
Blue: correct
Red: too strong
5. After you have performed the measurement, the result screen opens immediately.

Performing a measurement with Vivatmo pro

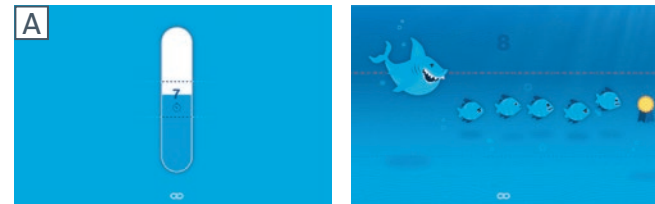


Visualization Scale

Visualization Fish

A Preparation:

1. Check whether the Vivatmo pro handheld is switched on.
2. On the **Home** screen, select the **Measure** button. Attach a disposable mouthpiece firmly onto the device.
3. Select an visualization and wait for the handheld to finish regeneration.

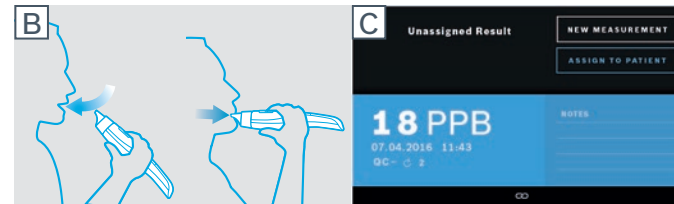

B Measuring:

The patient should sit in a relaxed position, inhale deeply through the mouth, lift the handheld to the mouth and exhale through the disposable mouthpiece. The breath control visualization helps the patient to stay within the dotted lines. On the display you can see the remaining time to the end of the measurement.

C Result:

After you have performed the measurement, the result screen opens immediately.

Managing measurements with Vivatmo pro



D Manage:


In the **Manage** menu, you can manage patient records, measurements, and orders.

Add new patients: On the **patients** tab, select **+Add new patient**.

Search Patients: Select the **Search** button and enter name or patient ID.

E Patient record:

In the patient file you can see all important information about the patient. You can edit, print, export, delete, or start a new measurement.

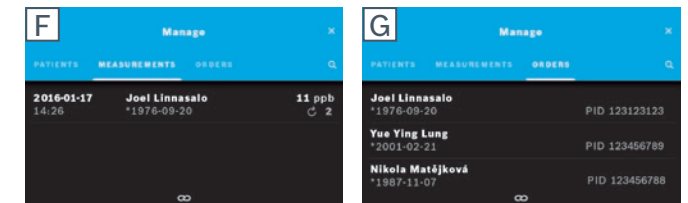

F Measurements:

Here you can select individual measurements and assign unassigned measurements to a patient.

G Orders:

Here you can see all current orders that you have sent to the device via the patient management system of your practice.