



BOSCH



vivatmo
pro

Administration instructions


Content

1	Administrator settings	4
2	Device settings	4
2.1	Setting the language	5
2.2	Setting date and time	5
2.3	Setting Auto-Lock	6
3	Quality Control (QC) settings	7
4	Printer configuration	7
5	Network and Interfaces.....	9
5.1	Handheld	9
5.1.1	Assigning the handheld	9
5.1.2	Release the handheld	10
5.2	Network configuration	11
5.2.1	Supported configuration of an ethernet connection	12
5.2.2	Supported installation of a WLAN connection	13
5.2.3	Manual ethernet configuration	14

5.2.4	Manual WLAN configuration	15
5.3	Order/result interface	16
5.4	Export and Backup file configuration	18
5.5	<i>Vivasuite</i> configuration	19
6	User administration.....	21
6.1	Add new user	22
6.2	Editing and reset user accounts	23
6.3	Enable/Disable user account	23
6.4	Deleting user accounts	24
7	Data backup.....	24

1 Administrator settings

These Administration instructions supplement the *Vivatmo pro* instructions for use explaining all administrator settings of the settings menu.

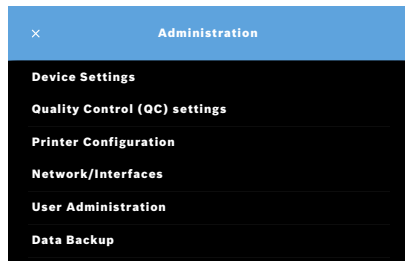


INFORMATION

Administration settings are activated only if you are logged in as administrator.


1. Log in as administrator.
2. In the Settings menu, select **ADMINISTRATION**.

2 Device settings

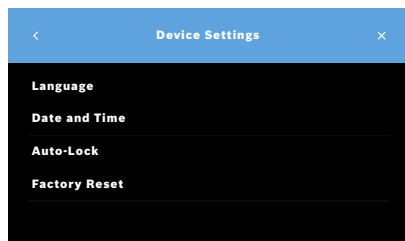

3. On the **Administration** screen, select **Device Settings**.
The **Device Settings** screen opens for changing Language, Date and Time, Auto-Lock or Factory Reset. Factory Reset resets all stored data and settings.


2.1 Setting the language

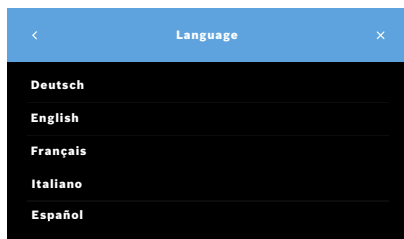

1. On the **Device Settings** screen, select **Language**.
2. Select the desired language.


2.2 Setting date and time

The time format is <dd/mm/yyyy>, <hh:mm> 0-24 h.

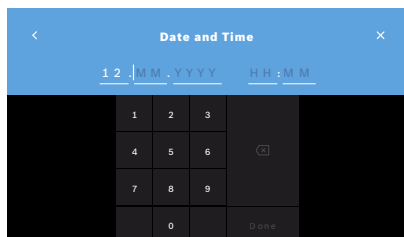

1. On the **Device Settings** screen, select **Date and Time**.
2. Enter day, month, year and the time.
3. Confirm with **DONE**.
4. Select the **CONTINUE** button to confirm your settings.

2.3 Setting Auto-Lock

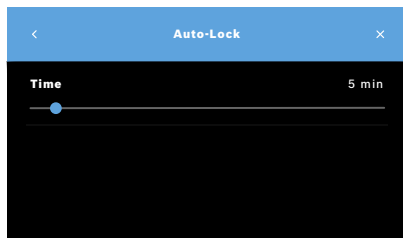

When Auto-Lock is activated, the system shuts down to save energy and protect patient data. The user needs to log in again after an automatic lockoff.

You can set the interval according to the needs in the clinical environment.

1. On the **Device Settings** screen, select **Auto-Lock**.
2. Select and hold the blue point and move it to the desired Auto-Lock time.

Factory reset

Factory reset removes all stored measurements from the basestation and all data of patients and users. Date and time are reset and no handheld will be connected afterwards.

INFORMATION

Use factory reset prior to disposal or service returns only.

3 Quality Control (QC) settings


External QC tests are used to control the measurements of the *Vivatmo pro* handheld in comparison to reference concentrations of nitric oxide (NO).



Quality control settings can only be accessed by administrators. The QC settings contain QC status, interval and number of measurements.

For further information see “6.2 Quality control (QC)” in General manual.

4 Printer configuration

1. On the **Administration** screen, select **Printer Configuration**.
The **List of Printers** screen opens. The standard printer is highlighted.
2. Select **Add new printer**.
3. Select the desired printer interface:
 - USB
 - Network
 - SharedSelect the **Right Arrow** to move to the **Select Printer** screen.

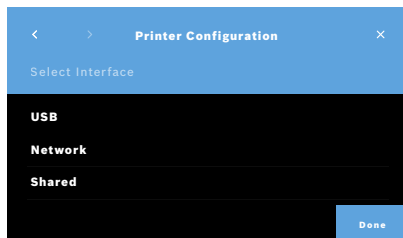

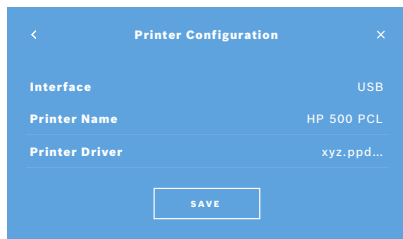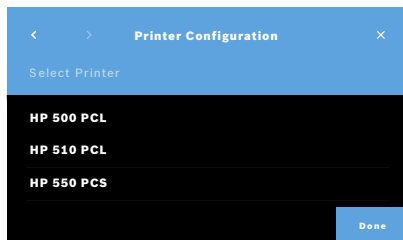

INFORMATION

Printing data is transmitted unencrypted.

4. On the **Select Printer** screen, select the desired printer.
5. Select the **Right Arrow** to move to the **Select Printer Driver** screen.
6. Select the **printer driver**.

The **Printer Configuration** screen is displayed.

7. Confirm with **SAVE** to save the printer configuration.
The list of printers is shown.


Elements in the list of printers:

1. Mark a printer to select your standard printer.
2. Select the **Delete** button to delete a printer from the list.
3. Select the **x** button to close the list.

5 Network and Interfaces

5.1 Handheld

5.1.1 Assigning the handheld

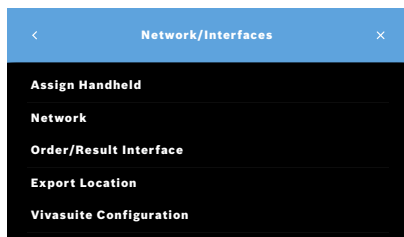

The screenshot shows a mobile application interface with a blue header bar containing a back arrow, a forward arrow, the text 'Handheld', and a button labeled 'SCAN FOR HANDHELDS'. Below the header, a table lists handheld devices with their names and serial numbers.

NAME	SERIAL NUMBER
Vivatmo pro	1234567890
Vivatmo pro	0987654321

1. On the **Administration** screen, select **Network/Interfaces**.
2. On the **Network** screen, select **Assign Handheld**.
3. Turn on your handheld.
4. Select **SCAN FOR HANDHELDS**.
5. Wait until the scan procedure is completed.
You see the list of active handhelds.
If no handheld is active, you see the message: "No handhelds available".
6. Compare the number of the *Vivatmo pro* handheld in the list with the serial number beneath the symbol **SN** on the handle of the handheld.
7. From the list, select the *Vivatmo pro* handheld which shall be installed with your *Vivatmo pro* basestation.
8. The Configuring device screen opens. The handheld and the basestation connect automatically.

The *Vivatmo pro* handheld and the *Vivatmo pro* basestation show the same date and time now.

INFORMATION

If QC is activated, perform a QC-measurement after assigning a new handheld. The basestation software stores the QC status of the handheld with each measurement.

5.1.2 Release the handheld

In case another handheld was installed before, it is disconnected by tapping the **Release Handheld** button. If you want to dispose the old handheld, clear memory and take out the battery.



1. Keep the **ENTER** button and the **Arrow right** button pressed for 3 seconds.



The **Clear memory** icon is selected.



2. Use the **Arrow** buttons to select the check mark.



3. Press the **ENTER** button to confirm the clear memory.



An animation is shown while the **Clear memory** icon flashes.


4. Press the **ENTER** button to return to the main screen.
5. Switch off the handheld.

5.2 Network configuration

Vivatmo pro can be connected to a local area network to receive measurement orders, send back measurement results, export patient related measurement data, or use a network printer. *Vivatmo pro* supports Ethernet and Wireless via WLAN connection.

INFORMATION

***Vivatmo pro* shall not be used in unprotected networks.** Data security using user credentials shall be used in accordance with the local data security policies. Data is transmitted unencrypted.

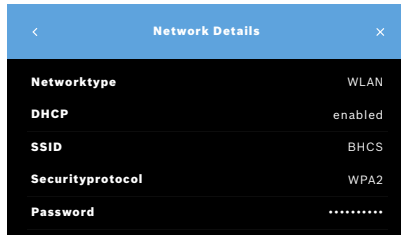

1. On the **Network/Interfaces** screen, select **Network**.



2. If a network access is configured already, select

Show Network Configuration.

The **Network Details** are:



- **Networktype:** WLAN or Ethernet
- **DHCP:** Dynamic Host Configuration Protocol
When DHCP is enabled, the device supports the configuration of the network.
- **SSID:** Service Set Identifier
Shows the name of the WLAN network.
- **Securityprotocol:** Shows the security protocol used for your system.
- **Password:** Shows the network password.

5.2.1 Supported configuration of an ethernet connection

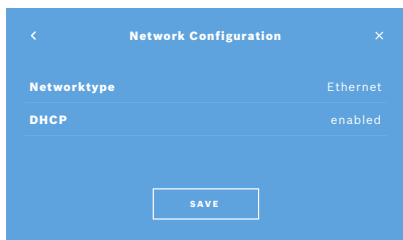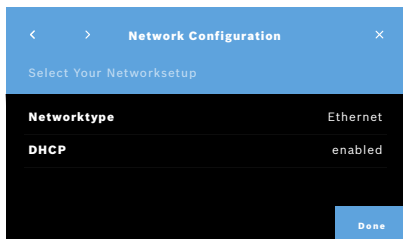




1. Plug in the ethernet cable in the ethernet port on the rear of the *Vivatmo pro* basestation.
2. On the **Network** screen, select **Configure Network**.
3. Select the Networktype **Ethernet** in the drop down menu.
4. Enable **DHCP**.
5. Confirm with **SAVE**.

5.2.2 Supported installation of a WLAN connection

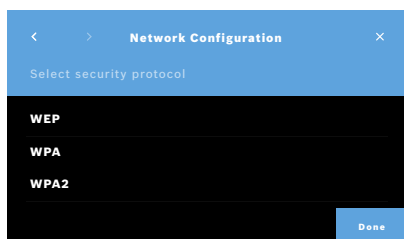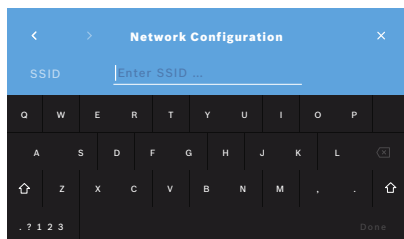

1. On the **Network** screen, select **Configure Network**.
2. Select the Networktype **WLAN**.
3. Enable **DHCP**.
4. On the **Set network SSID** screen, enter the name of the WLAN.
5. Select the **Right Arrow** or the **DONE** button to leave the screen.
6. Select the security protocol activated in your WLAN.
7. Select the **Right Arrow** button to leave the screen.
8. On the next screen, enter the WLAN Password of your WLAN.
9. Select the **Right Arrow** or confirm with **DONE** to leave the screen.
10. Confirm with **SAVE**.

5.2.3 Manual ethernet configuration

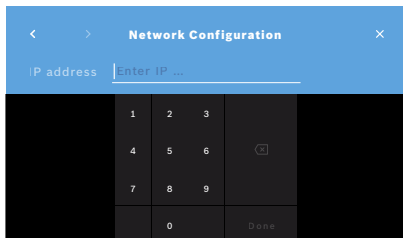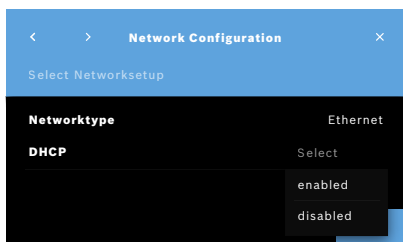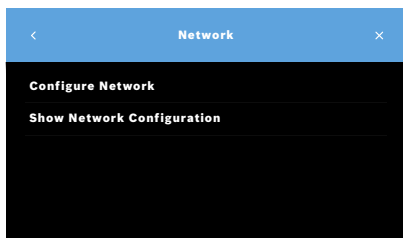

1. Plug in the ethernet cable in the ethernet port of the basestation.
2. On the **Network** screen, select **Configure Network**.
3. Select the **Networktype Ethernet**.
4. Disable **DHCP**.
5. Enter the **IP-Address** of your network and the Subnet Mask of your network.
6. Enter the **Gateway ID**.
7. Select the **Right arrow** or confirm with **DONE** to leave the screen.

Network Configuration

Networktype	Ethernet
DHCP	enabled
IP address	- . - . - . -
Subnet Mask	- . - . - . -
Gateway	- . - . - . -
SSID	BHCS
Securityprotocol	WPA2
Password

SAVE

8. Confirm with **SAVE**.


5.2.4 Manual WLAN configuration

1. On the **Network** screen, select **Configure Network**. The **Network Configuration** screen opens.
2. Select the Networktype **WLAN**.
3. Disable **DHCP**.
4. Follow steps 4 to 8 of “5.2.2 Supported installation of a WLAN connection”.
5. Follow step 5 to 8 of “5.2.3 Manual ethernet configuration”.
After a successful configuration, the **Network Configuration** screen is displayed.
6. Confirm with **SAVE**.

5.3 Order/result interface

Vivatmo pro can receive measurement orders from and send results to an electronic patient record system (EPR), if the network is configured. The interface standards GDT (German Data Transfer) and HL7 are supported.

To configure the interface to your patient management system follow the installation wizard:

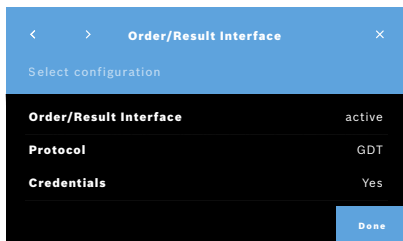

1. On the **Administration** screen, select **Network/Interface**.
2. On the **Network and Interface** screen, select **Order/Result Interface**.
You can select:
 - **Configure Interface** to change the settings.
 - **Show Interface Configuration** to display the current settings.

Select **Configure Interface** to change settings.

3. Activate the **Order/Result Interface**. The system keeps the interface settings when deactivated.
4. Select the **Arrow Right** button to move to the next screen. Select the desired protocol standard: **HL7** or **GDT**.
5. Select **Yes** for Credentials when the **Order/Result Interface** is access controlled by user name and password.

Order/Result Interface

Inbox

Q W E R T Y U I O P
A S D F G H J K L
Z X C V B N M , . ' " ~

Done

Order/Result Interface

Username

Q W E R T Y U I O P
A S D F G H J K L
Z X C V B N M , . ' " ~

Done

Order/Result Interface

Order/Result Interface active

Protocol GDT

Inbox location smb://example/vivatmo/inbox

Outbox smb://example/vivatmo/outbox

Username Praxis-admin-1

Password

Communication Test Positive

SAVE

6. In accordance to IT definitions in your organization, define the location for the inbox for incoming measurement orders.
7. Repeat the procedure for the location of the outbox.
8. Confirm with **DONE**.

When credentials are required, follow steps 9 to 13.

9. Enter the user name required to access the data from the EPR system.
10. Confirm with **DONE** to move to the next screen.
11. Enter the password required to access the data from the EPR system.
12. Confirm with **DONE**.

The settings for the **ORDER/RESULT INTERFACE** are displayed including the result of a Communication Test.



13. Check the data and confirm with **SAVE** when the data are correct. On your home screen, you see the order button, see "2.1 Screen elements" in General manual.

If you delete the inbox and the outbox, the order button disappears from the home screen.


5.4 Export and Backup file configuration

You can export data from patient records (see “5.1 Patient records” in General manual) or backup the entire measurement and patient data (see “7 Data backup”) to a defined storage location. The storage location can rather be a USB drive or a network drive. Export files of a patient can be identified by the time stamp and the patient name. Export data are transmitted unencrypted. The backup file is stored encrypted and can be imported to the same basestation only.

For configuration follow the configuration wizard:

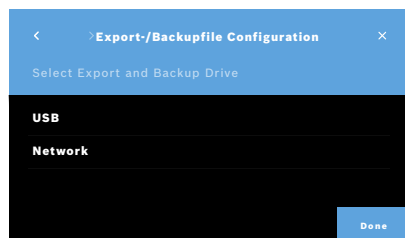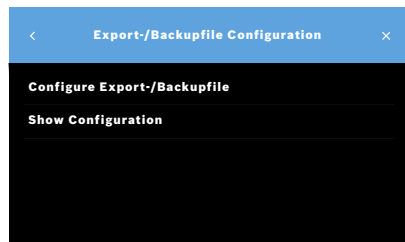

1. On the **Administration** screen, select **Network and Interfaces**.
2. On the **Network and Interfaces** screen, select **Location for Export/Backup**.
3. Select **Configure Export/Backup File** to change the settings.
4. In accordance to IT definitions in your organization, define the location for the storage of the export and backup files on USB or network drive. End the location path always with “/”-symbol.

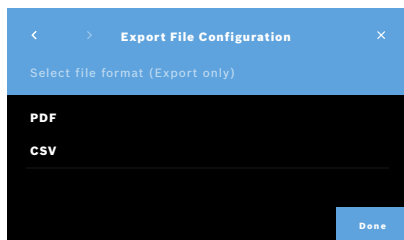


5. Select **CSV** or **PDF** as export format.
The settings for the **Location for Export/Backup** are displayed.
6. Check the data and confirm with **SAVE** button when the data are correct.

5.5 Vivasuite configuration


Vivasuite is the digital *Vivatmo* ecosystem allowing to use services such as device management and remote software update. *Vivasuite* runs in the Bosch IoT Cloud and applies highest standards regarding IT security and data privacy.

A registration to *Vivasuite* and pairing of your device is required. Your device must be connected to an internet- enabled network.

To connect *Vivatmo pro* to *Vivasuite* perform the following steps:

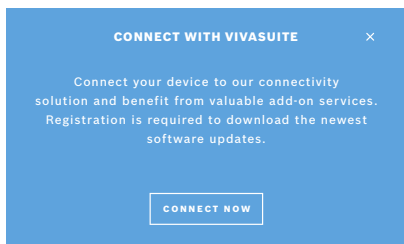


1. *Vivasuite*: Register an account on www.vivasuite.com and log into your account.
2. *Vivatmo pro*: On the Administration Screen select **Network/Interfaces**. On the **Network/Interfaces** screen, select **Vivasuite configuration**.
3. *Vivatmo pro*: Click through introduction and configure your network connection (for details see “5.2 Network configuration”) until devices show screen with the pairing code.

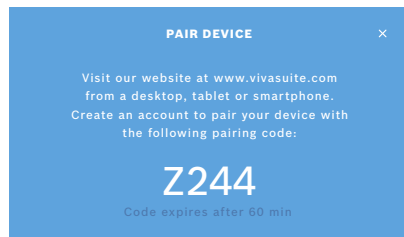


4. *Vivasuite*: Click **Add device**.
5. *Vivasuite*: Enter the pairing code shown on the *Vivatmo pro*.
6. *Vivatmo pro*: Confirmation screen is shown.

To disconnect *Vivatmo pro* from *Vivasuite* perform the following steps:

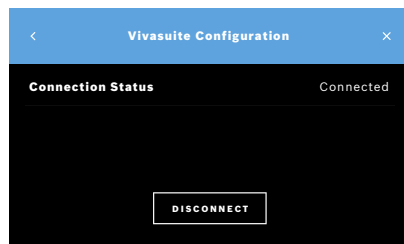


1. *Vivatmo pro*: On the **Network/Interfaces** screen, select **Vivasuite Configuration**.
2. *Vivatmo pro*: Select **Disconnect**. You are asked to confirm before device gets disconnected.

6 User administration

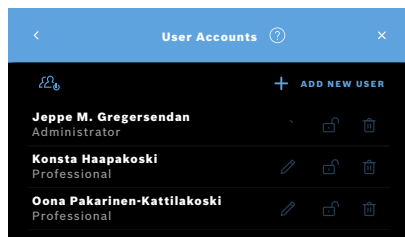

1. On the **Administration** screen, select **User Administration**.

Access control by user accounts can be disabled by the button .

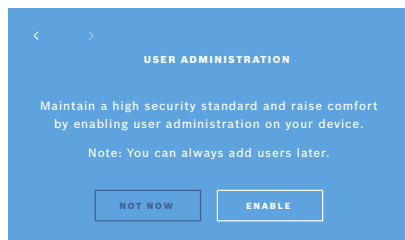

When **USER ADMINISTRATION** is not enabled in the setup procedure, the **User Accounts** screen opens to **ENABLE** individual user accounts.

INFORMATION



By disabling the User Administration the user settings are kept for later use. See also “3.2.1 User concept” in General manual.

INFORMATION


The system needs at least 1 administrator login. If the administrator passcode is forgotten, *Vivatmo pro* needs to be reset by a service technician.

6.1 Add new user

For adding a new user, follow the configuration wizard:

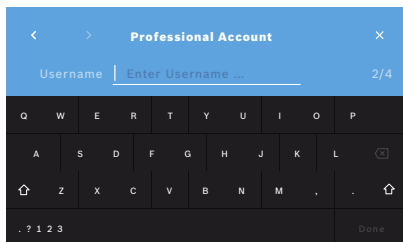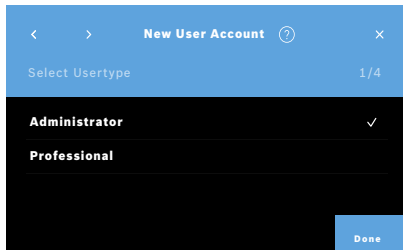

1. On the **User Accounts** screen, select the **+ ADD NEW USER** button.
2. Select the desired **user type**, see “3.2.1 User concept” in General manual:
 - Administrator
 - Professional
3. Enter the **name** of the user.

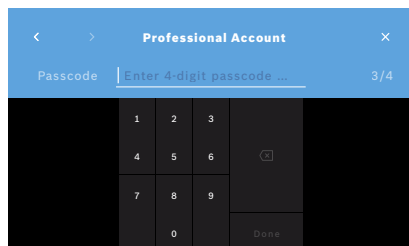

4. Enter the respective **passcode**:
 - a 4 digit passcode for a professional user.
 - an 8 digit passcode for an administrator account.
5. Repeat the passcode.
6. Confirm with **DONE**.
The user is successfully added.

6.2 Editing and reset user accounts



1. On the **User Accounts** screen, select the user you want to edit.
2. Select the **Pencil** button.
3. Enter your changes to user name and passcode.
4. Confirm with **DONE**.

6.3 Enable/Disable user account



1. On the **User Accounts** screen, select the user you want to enable/disable.
2. Select the **Enable/Disable user** button.
When lock is open, the user access is enabled.


6.4 Deleting user accounts



1. On the **User Accounts** screen, select the user you want to delete.
2. Select the **Delete** button.
The account is deleted.

7 Data backup

Data backup stores the entire measurement and patient data encrypted at the defined storage location, see “5.4 Export and Backup file configuration”. Due to the encryption mechanism the Backup data can be imported to the same basestation only. The import of the backup overwrites the data stored on the basestation.

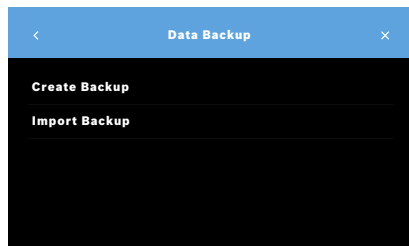

1. On the **Administration** screen, select **Data Backup**.
2. Select between:
 - **Create Backup**
 - **Import Backup**The backup procedures might take up to 5 minutes.



เครื่องวิทยุคมนาคมนี้ ได้รับยกเว้น ไม่ต้องได้รับ
ใบอนุญาตให้มี ใช้อุปกรณ์วิทยุคมนาคม
หรือตั้งสถานีวิทยุคมนาคมตามประกาศ กสทช.
เรื่อง เครื่องวิทยุคมนาคม และสถานีวิทยุ
คมนาคมที่ได้รับยกเว้นไม่ต้องได้รับใบอนุญาต
วิทยุคมนาคม ตามพระราชบัญญัติวิทยุ
คมนาคม พ.ศ. 2498



nanb. | โทรคมนาคม
คำศัพท์และเงื่อนไขการใช้งาน
Call Center 1200 (InSW5)

OMAN - TRA
TA-R/10512/20
D172338


TRA
REGISTERED No.
ER92285/20
DEALER No.
DA97534/20

Complies with
IDMA standards
DA105282

72413/SPD-
DI/2021
10325

72412/SPD-
DI/2021
10325

Oxycap

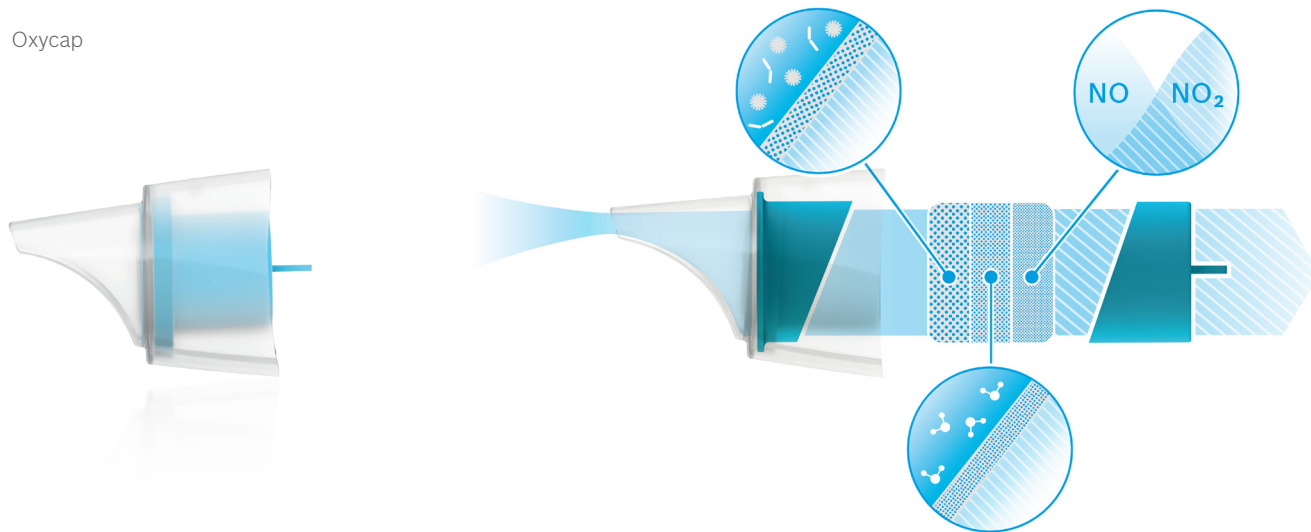

BOSCH

Bosch Healthcare Solutions GmbH
Stuttgarter Straße 130
71332 Waiblingen
Germany



F09G100560_02
Administration instructions
Latest change:
2021-06-11



Bosch Healthcare Solutions GmbH
Stuttgarter Straße 130
71332 Waiblingen, Germany