


First steps with your Vivatmo pro



- ① Handheld
- ② Charging cradle
- ③ On/Off button
- ④ Touchscreen
- ⑤ Power adapter port

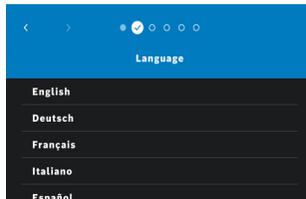

1. Carefully insert the rechargeable battery in the battery compartment of the handheld.
2. Connect the basestation with the power supply. The basestation then starts automatically.
3. Remove the protective cap and place the handheld on the charging cradle for a few hours.
4. Follow the instructions on the Vivatmo pro screen.



Please consider the following before using Vivatmo pro: Take time to carefully read the instructions for use before using the measurement system.

This quick guide does not replace the instructions for use. Inspect the basestation, the handheld, the disposable mouthpiece, and its pouch before usage. If you notice any damage, do not use the device or accessories.

Invented for life



vivatmo-hcp.com



Bosch Healthcare Solutions GmbH
Alte Bundesstraße 50
71332 Waiblingen
Germany



F09G100424 Rev 3.0
Latest change: 2025-05-28


vivatmo
pro
Quick guide

Vivatmo pro – Screen elements



- ① Measurement orders from electronic patient record (EPR) system
- ② Manage patient measurements and orders
- ③ Charging status handheld battery
- ④ Status of handheld connection
- ⑤ Start measurement
- ⑥ Settings menu/logout

Perform a FeNO measurement

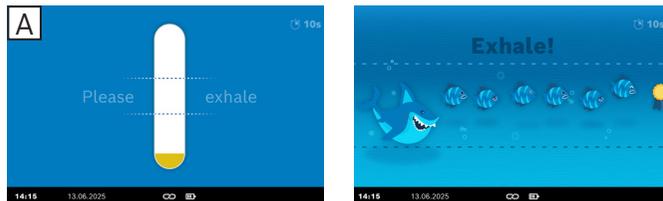


Scale visualization

Fish visualization

A Preparation:

1. Check that your Vivatmo pro handheld is switched on and connected to the basestation.
2. On the **Home screen**, select the **MEASURE** button.
3. Select measurement mode. The regular 10 seconds is pre-selected. Shortened 6 seconds can be selected for children (7-11 years). Press **CONTINUE**.
4. Attach a disposable mouthpiece firmly onto the handheld.
5. Select a visualization and wait for the handheld to finish regeneration.

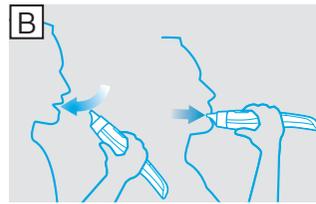

B Measuring:


The patient should sit in an upright position, inhale deeply through the mouth, lift the handheld to the mouth, and exhale through the disposable mouthpiece. The visualization helps the patient to stay within the dotted lines. On the display you can see the remaining time to the end of the measurement.



C Result:

After you have performed the measurement, the result screen opens immediately. If the measurement has failed, this will be indicated on the screen. Press the **ENTER** button on the handheld and repeat the measurement.

Manage measurements

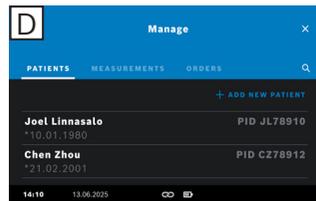


D Manage:

In the **Manage** menu, you can manage patient records, measurements, and orders.


Add new patients: On the **PATIENTS** tab, select **+ADD NEW PATIENT**.

Search Patients: Select the **Search** button and enter name or patient ID.

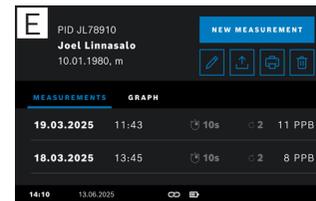


E Patient record:

In the patient file you can find the measurements assigned to a patient. You can select buttons to **EDIT**, **EXPORT**, **PRINT**, or **DELETE** patient data, or start a **NEW MEASUREMENT**.

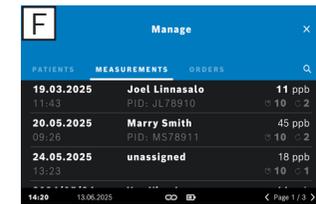

F Measurements:

Here you can select individual measurements and assign unassigned measurements to a patient.

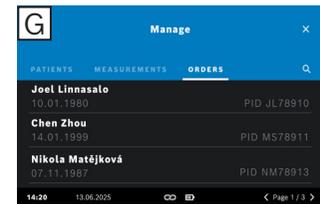


G Orders:

Here you can see all current orders that you have sent to the device via the EPR of your practice.

Measurement stand-alone with the handheld

Stand-alone measurements can be done when the handheld has no connection to the basestation, when the basestation is out of range or switched off.

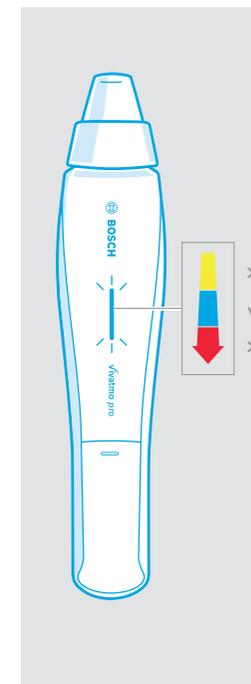

1. Check whether the Vivatmo pro handheld is switched on and attach a new disposable mouthpiece.
2. Press the **Enter** button and wait for the handheld to finish regeneration.
3. When the display shows the **Breathing icon**, start the measurement.
4. The color of the **LED** provides feedback on the correct breathing strength: **Yellow:** too weak
Blue: correct
Red: too strong
5. After you have performed the measurement, the result screen opens. Results are synchronized after placing the handheld device back onto the charging cradle.